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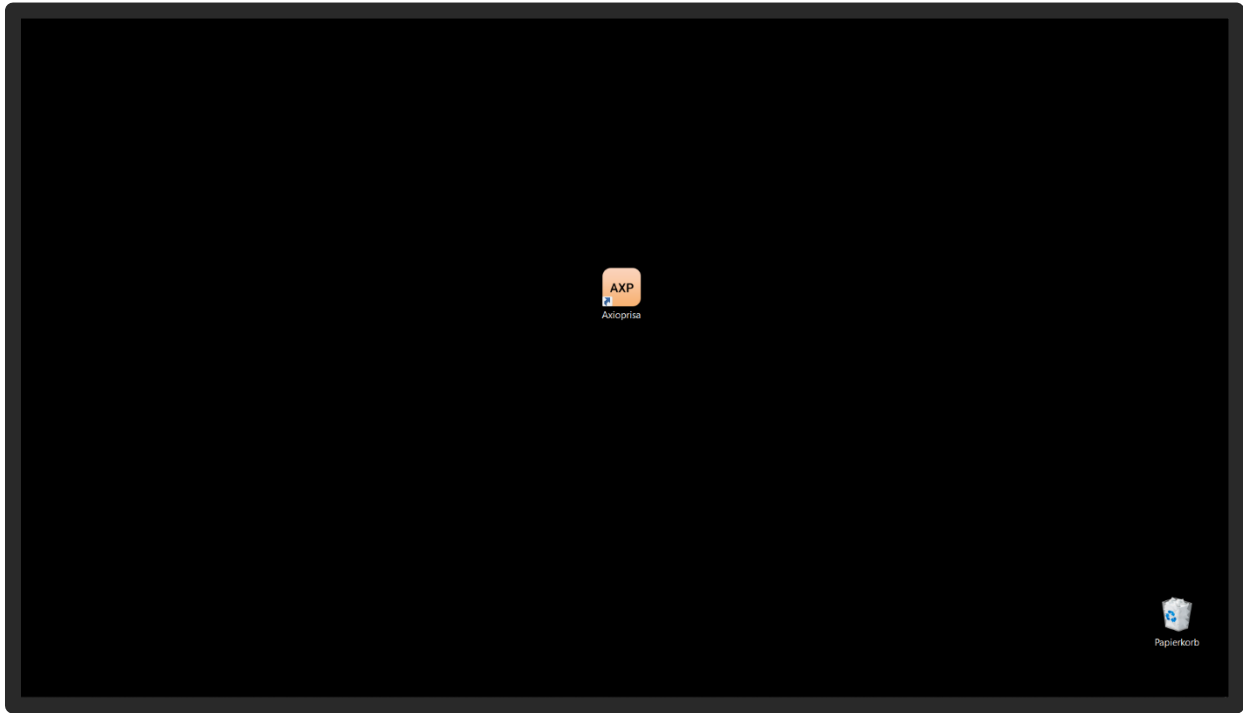
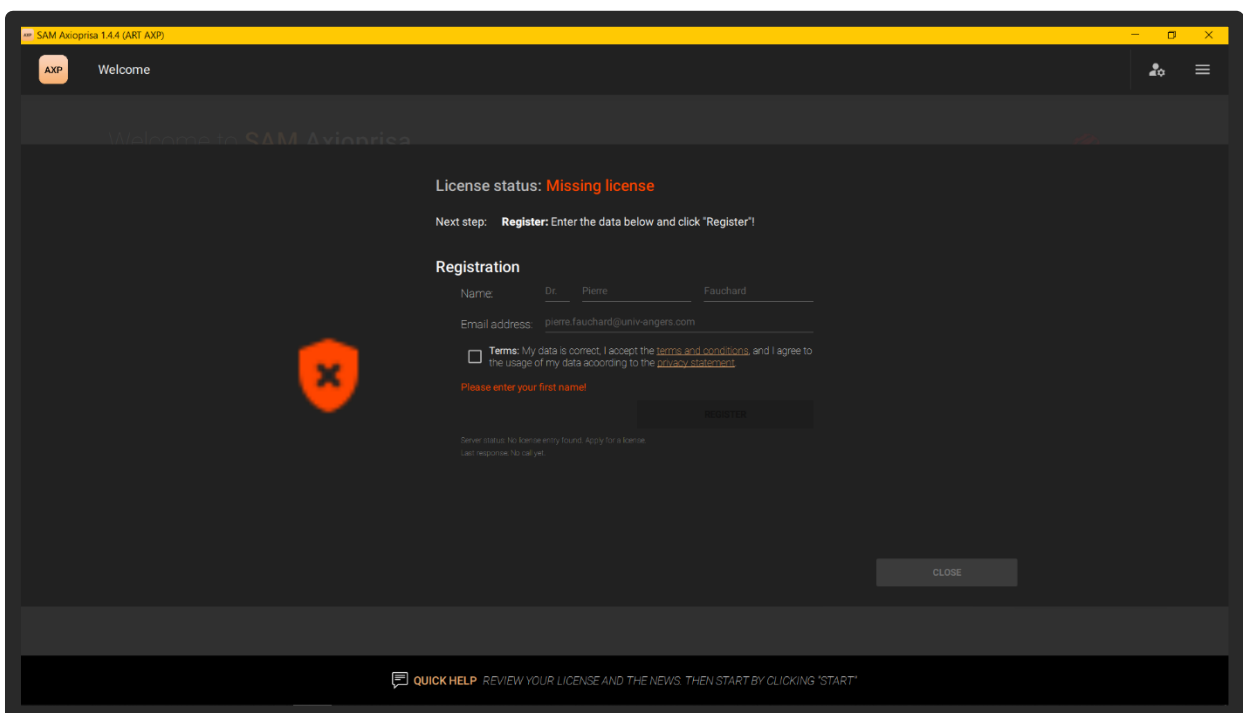
*After installing the software, the licence key received must be entered in order to use the software. To do this, open the application by double-clicking on the link.*

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
*Registration is necessary for activation. For this purpose, the name of the practitioner or the laboratory owner is entered and the e-mail address of the company is stored.*

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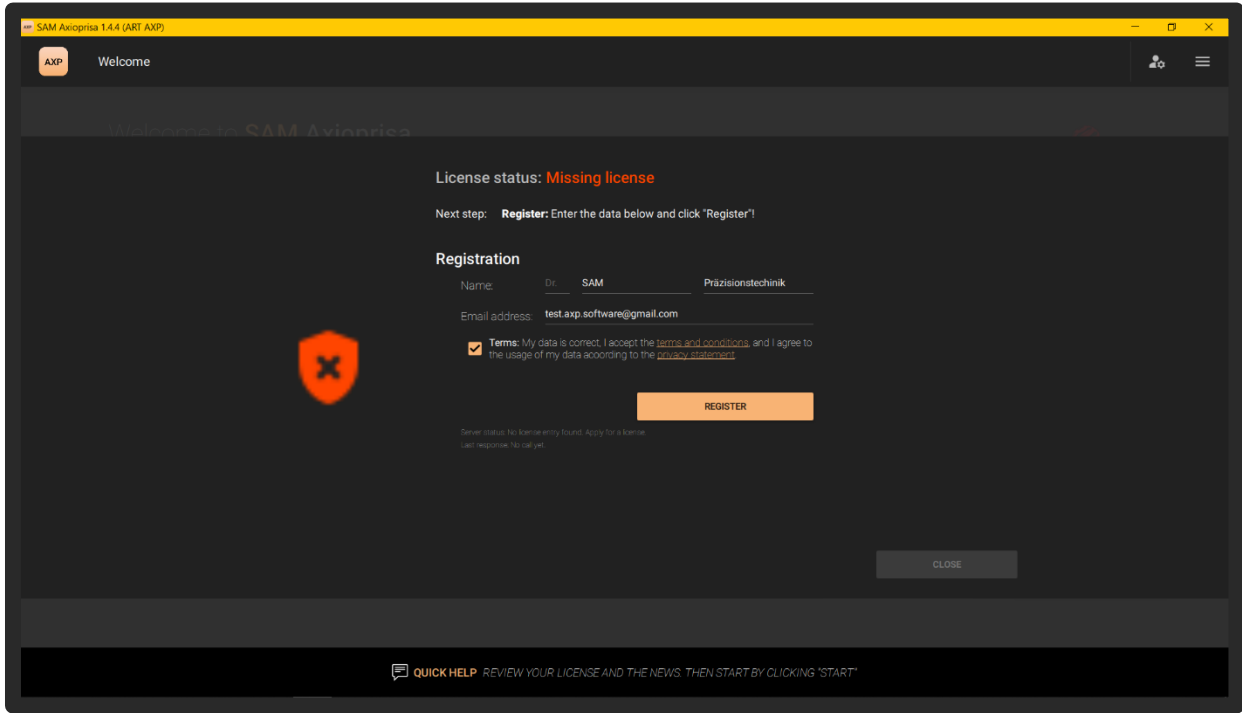
*The General Terms and Conditions and the Privacy Policy should be read carefully and confirmed with the check mark next to them. Then click on "REGISTER" to complete the registration.*

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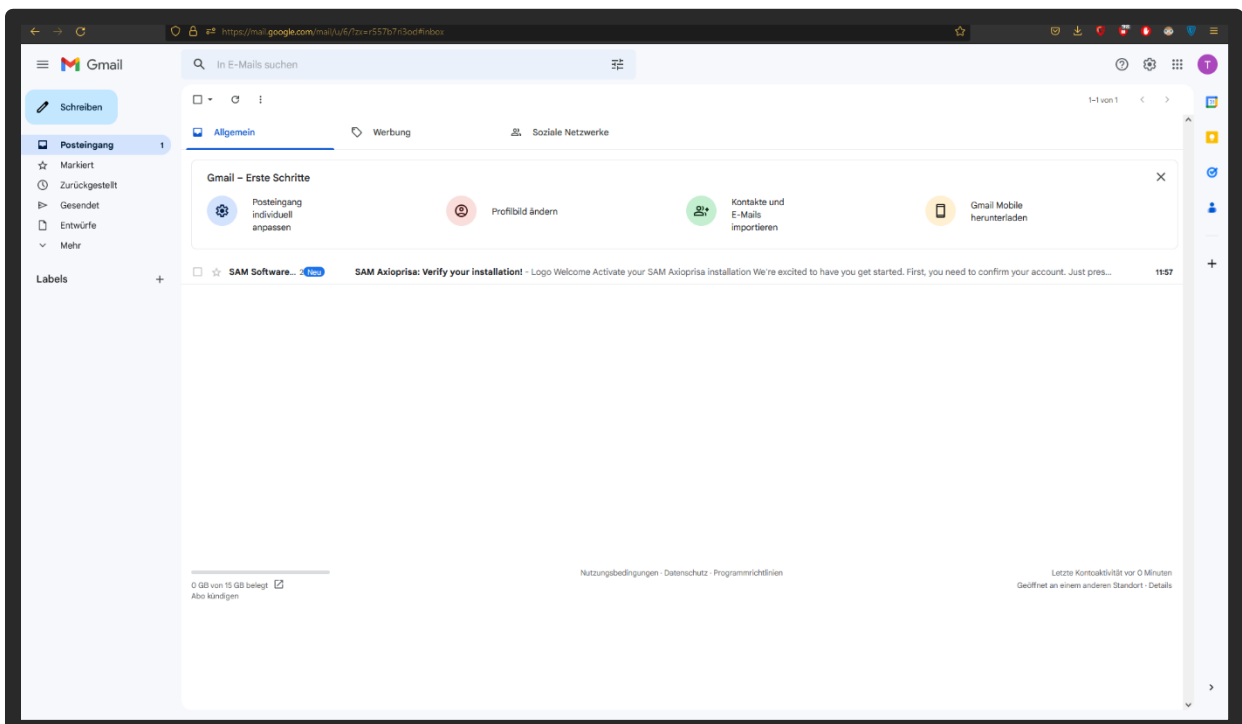
*You receive a confirmation email with a link. If this mail is not displayed in the email box after 10 minutes, you should check your spam folder.*

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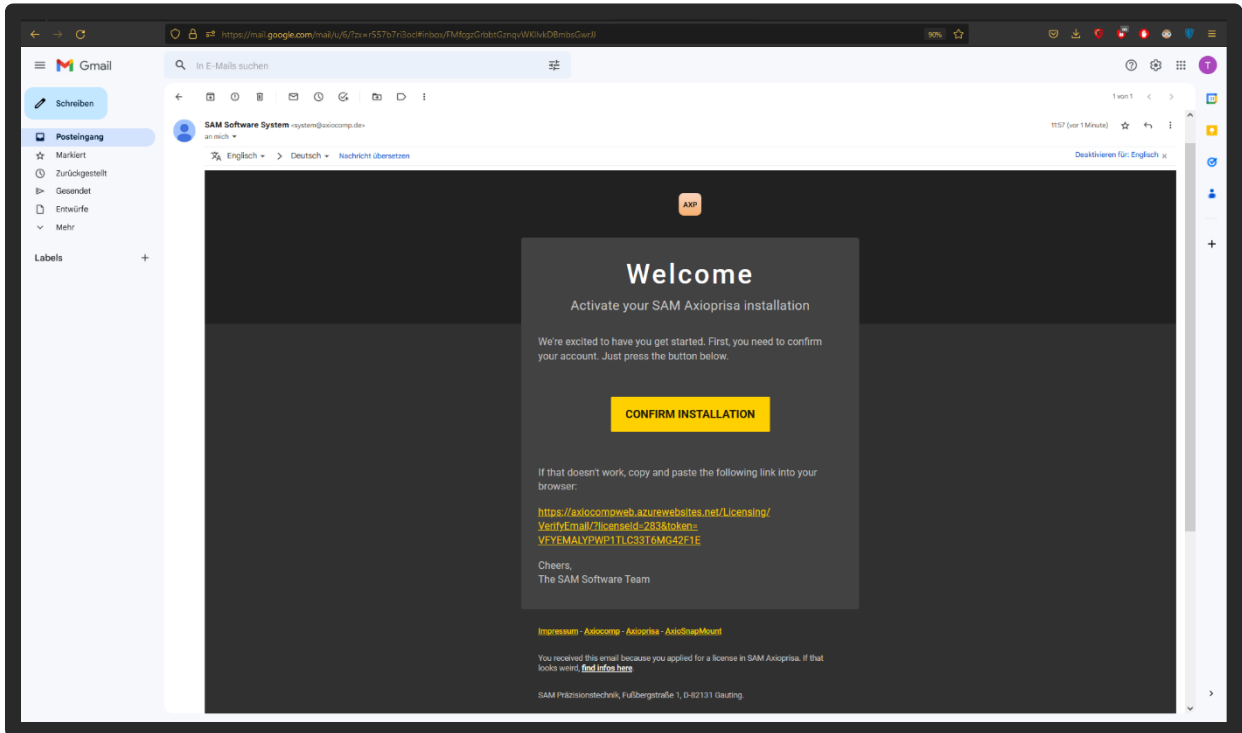
*The email contains an activation link that must be confirmed.*

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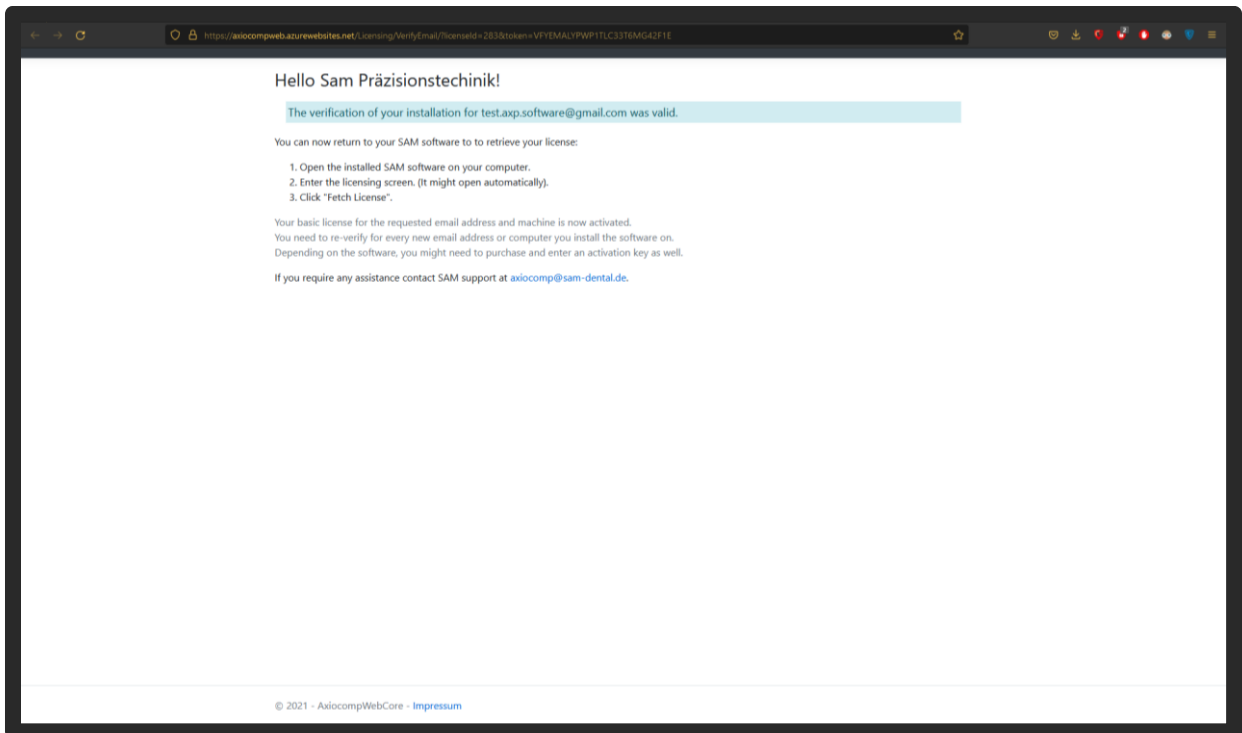*After confirming the link, you will be redirected to a confirmation page. Now you can return to the software.*

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*After confirming the registration, another area appears for entering the licence key. The licence key is on the licence card that was sent by post.*

The screenshot shows the SAM AxioPrisa 1.4.4 (ART AXP) software interface. The license status is "Not activated yet". The next step is to activate the product by entering a fresh license code. The registration form is filled with "Dr. Sam Präzisionstechnik" and "test.axp.software@gmail.com". The "UPDATE LICENSE" button is visible. The activation section shows a license key field with "6MFCJP2" and a "REDEEM KEY" button.

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
*Uncover the licence key on the card and enter the key in the corresponding field. Confirm the entry with "REDEEM KEY".*

The screenshot shows the SAM AxioPrisa 1.4.4 (ART AXP) software interface. The license status is "Not activated yet". The next step is to activate the product by entering a fresh license code. The registration form is filled with "Dr. Sam Präzisionstechnik" and "test.axp.software@gmail.com". The "UPDATE LICENSE" button is visible. The activation section shows a license key field with "7QBR7P1" and a "REDEEM KEY" button.

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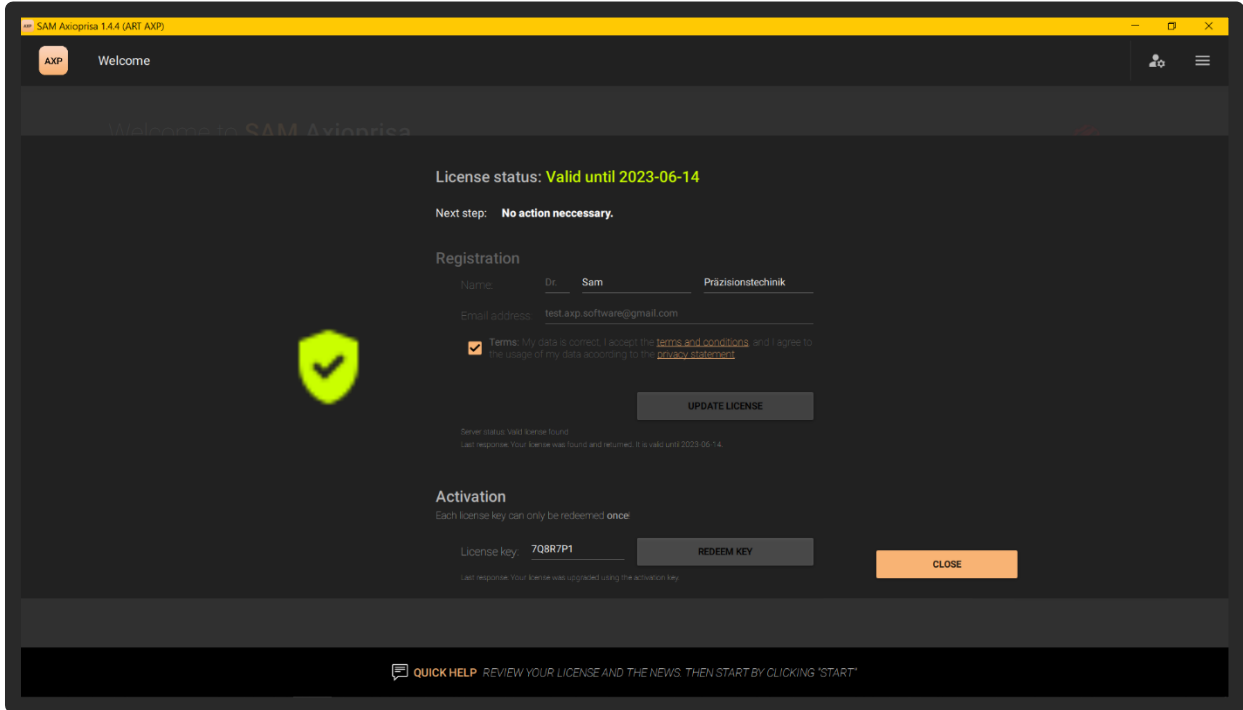*If the licence key is valid, the validity date is shown in green at the top. The window can be closed with "CLOSE".*

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*Now the software can be used. The validity date is always shown in green letters in the upper left-hand area of the start screen.*